**SHIELD**  
THE WORLD.

Pipeline Flex™  
With **SHIELD**  
Technology™

**ACCEPT NO SUBSTITUTE  
IN CLINICAL EVIDENCE**

PIPELINE™ EMBOLIZATION DEVICE\*

**PEDESTRIAN Registry**  
Long-term outcomes from the  
**Pipeline™** Embolization Device  
for the treatment of Intracranial  
Aneurysms

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# Pipeline Embolization Devices for the Treatment of Intracranial Aneurysms, Single-Center Registry: Long-Term Angiographic and Clinical Outcomes from 1000 Aneurysms

Ivan Lylyk, MD   
Esteban Scrivano, MD   
Javier Lundquist, MD  
Angel Ferrario, MD  
Carlos Bleise, MD   
Nicolas Perez, MD  
Pedro Nicolas Lylyk, MD   
Rene Viso, MD   
Rodolfo Nella-Castro, MD  
Pedro Lylyk, MD 

Department of Interventional Neuro-radiology and Neurosurgery, Instituto Médico ENERI—Clínica La Sagrada Familia, Buenos Aires, Argentina

**Correspondence:**  
Ivan Lylyk, MD,  
Instituto Médico ENERI—Clínica La Sagrada Familia,  
Jose Hernandez 1642,  
Buenos Aires, Argentina.  
Email: [ivan@lylyk.com.ar](mailto:ivan@lylyk.com.ar)

**Received,** November 24, 2020.  
**Accepted,** March 31, 2021.

© Congress of Neurological Surgeons  
2021.

This is an Open Access article distributed  
under the terms of the Creative

**BACKGROUND:** Prospective studies have established the safety and efficacy of the Pipeline™ Embolization Device (PED; Medtronic) for treatment of intracranial aneurysms (IA).

**OBJECTIVE:** To investigate long-term outcomes from the Pipeline Embolization Devices for the Treatment of Intracranial Aneurysms (PEDESTRIAN) Registry.

**METHODS:** The PEDESTRIAN Registry data were retrospectively reviewed, which included patients (March 2006 to July 2019) with complex IAs treated with PED. Patients with unfavorable anatomy and/or recurrence following previous treatment were included and excluded those with acute subarachnoid hemorrhage. The primary angiographic endpoint was complete occlusion and long-term stability. Clinical and radiological follow-up was performed at 3 to 6 mo, 12 mo, and yearly thereafter.

**RESULTS:** A total of 835 patients (mean age  $55.9 \pm 14.7$  yr; 80.0% female) with 1000 aneurysms were included. Aneurysms varied in size: 64.6% were small ( $\leq 10$  mm), 25.6% were large (11–24 mm), and 9.8% were giant ( $\geq 25$  mm). A total of 1214 PEDs were deployed. Follow-up angiography was available for 85.1% of patients with 776 aneurysms at  $24.6 \pm 25.0$  mo (mean). Complete occlusion was demonstrated in 75.8% of aneurysms at 12 mo, 92.9% at 2 to 4 yr, and 96.4% at  $>5$  yr. During the postprocedural period, modified Rankin Scale scores remained stable or improved in 96.2% of patients, with stability or improvement in 99.1% of patients  $>5$  yr. The overall major morbidity and neurological mortality rate was 5.8%.

**CONCLUSION:** This study demonstrated high rates of long-term complete aneurysm occlusion, stable or improved functional outcomes, and low rates of complications and mortality. Clinical and angiographic outcomes improved over long-term follow-up, demonstrating that endovascular treatment of IA with PED is safe and effective.

**KEY WORDS:** Flow diverter, Follow-up studies, Intracranial aneurysm, Stroke, Subarachnoid hemorrhage, Pipeline Embolization Device

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## Pipeline™ Shield Subgroup vs Overall PEDESTRIAN Results

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Pipeline™ Embolization Device†

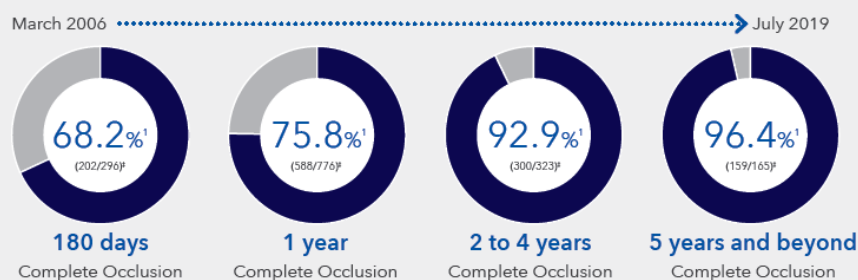
## PEDESTRIAN REGISTRY

Long-term outcomes from the Pipeline™ Embolization Device† for the treatment of Intracranial Aneurysms

PEDESTRIAN is the largest long-term study to date on Pipeline™, including 1,000 aneurysms treated.†  
This study demonstrated:

- High rates of long-term complete occlusion
- Stable or improved functional outcomes
- Low rates of complications and mortality

**Major Morbidity & Mortality (M&M):**  
5.8% over entire study period¹  
**Complete Occlusion**

### Pipeline™ Flex with Shield Technology™ subgroup showed:

(Pipeline™ Shield Subgroup⁵  
vs. Overall PEDESTRIAN Results)

- Lower complication rates overall (M&M: **2.4%** vs 5.8%)²
- Similar complete occlusion rates (92.5% @ 2yrs vs 92.9% @ 2-4yrs)²
- Trends towards less thromboembolic complications²

[View full article](#)

† Includes Pipeline™ (Classic) Embolization Device (no longer commercially distributed), Pipeline™ Flex Embolization Device & Pipeline™ Flex Embolization Device with Shield Technology™

² Patient numbers at follow up

³ n=274 aneurysms (213 patients of which 80 patients were included in PEDESTRIAN)

## Subgroup showed: ²

(Pipeline™ Shield Subgroup\*\*\* vs. Overall PEDESTRIAN Results)

- ▶ Lower complication rates (M&M: 2.4% vs. 5.8%)
- ▶ Similar complete occlusion rates (92.5% @ 2yrs vs. 92.9% @ 2 to 4yrs)
- ▶ Trends towards less thromboembolic complications

<Reference>

1. Neurosurgery 0:1–7, 2021, <Pipeline Embolization Devices for the Treatment of Intracranial Aneurysms, Single-Center Registry: Long-Term Angiographic and Clinical Outcomes from 1000 Aneurysms>
2. Lylyk, I. Long-term outcomes from the Pipeline Embolization Device for the Treatment of Intracranial Aneurysms (PEDESTRIAN) registry with PED Shield Sub-analysis. Presented at Society of Neurointerventional Surgery Conference. Colorado Springs, CO, USA. July 2021.

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