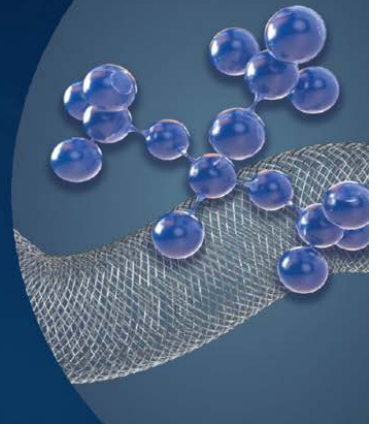
Pipeline™ Flex Shield Effect The Clinical Evidence

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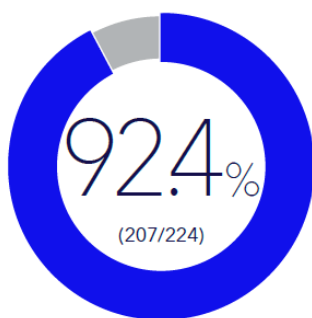
SCOPE-AUS Study : Safety and Clinical Effectiveness of Pipeline Shield Device for Intracranial Aneurysms

Chia GS, Rice H. et al. Stroke Vasc Interv Neurol. June 2022

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- **Complete Occlusion Rate (1 year)**

*included complex aneurysms like posterior circulation, distal small vessel, and bifurcation aneurysms.


- **0.7% Mortality and 2.0% Morbidity in ICA aneurysm**


"Lower than previously reported studies and suggest a potential benefit of the Shield Technology."

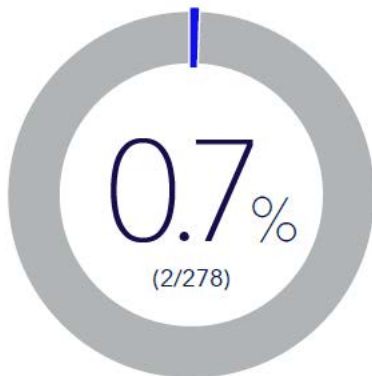- **Low treatment rate through 18 months or last follow-up**

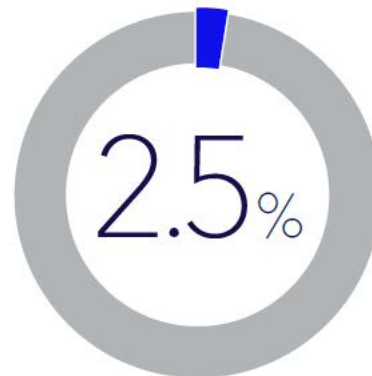- **Symptomatic ischemic event rate, with SAPT use after 3 months of DAPT**

† The authors are comparing to previously published studies of Trivelato et al., PFLEX, PEDSU, and SHIELD.

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"Surface modification may improve the safety profile during FDs treatment of these and other intracranial aneurysms by reducing stent thrombogenicity and potentially also reducing the need for prolonged dual antiplatelet therapy"

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Pipeline™ Flex with Shield Technology™ Placement for Small- and Medium-sized Aneurysms Achieves Early Complete Occlusion

Arai, S. et al., Neurologia medico-chirurgica. 20 Dec 2022

Study design	# of patients	Device	Time period	Aneurysms studied
Retrospective, single-arm, single center	N=128; 102 patients with 104 aneurysms were included for analysis; 26 patients did not meet the inclusion criteria.	<ul style="list-style-type: none">• Pipeline™ Classic (1%),• Pipeline™ Flex (51.9%),• Pipeline™ Shield (47.1%)	July 2013 to October 2021	Unruptured wide-necked aneurysms 5-12mm in size

Outcomes:

- Efficacy at 1 year follow-up using Raymond-Roy classification)
 - Complete occlusion: **92.3%** (96 aneurysms)
 - Residual neck 1% (1 AN), and residual AN 6.7% (7 aneurysms)
- Safety
 - **Low incidence of patients** with significant in-stent stenosis during the anatomical follow-up period
 - **Low incidence of procedural-related complications nor major strokes** during the clinical follow-up period
 - **Low incidence of patients** with worsening mRS score

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<Reference>

1. Chia, G. S., De Villiers, et al. Safety and clinical effectiveness of pipeline shield device for intracranial aneurysms in an Australian cohort (SCOPE-AUS). Stroke: Vascular and Interventional Neurology, 2(5). 2022. doi: 10.1161/svin.121.000292.
2. Arai S, Fujii T, et al. Pipeline Flex with Shield Technology Placement for Small- and Medium-sized Aneurysms Achieves Early Complete Occlusion. Neurol Med Chir (Tokyo). 2023 Feb 15;63(2):58-64. doi: 10.2176/jns-nmc.2022-0217.

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